



1. Sizing the Cushion

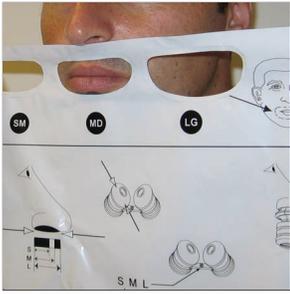


Place the closed mouth into the cut-outs on the product bag to determine the appropriate size.

2. Sizing the Pillow

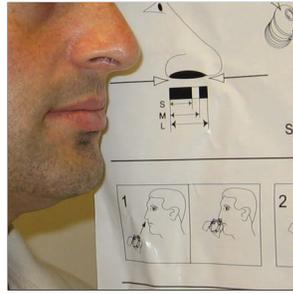

Measure the nose against the diagram on product bag to select size. An appropriate size should seal against the nostril rim.

3. Choosing the Appropriate Pillow Height Position

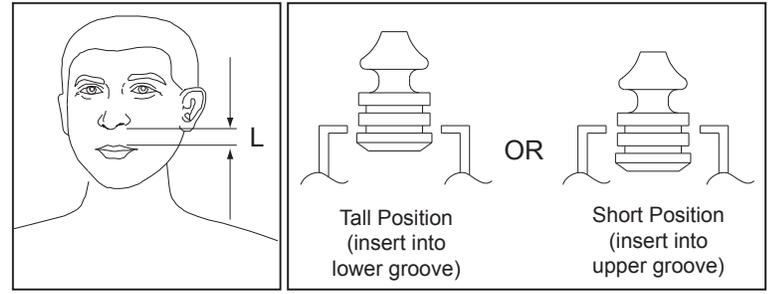


Measure Upper lip (L) distance:

$L > 0.3"$ (7.6 mm) : Use TALL position

$L < 0.3"$ (7.6 mm) : Use SHORT position

Insert pillows as shown into the corresponding grooves on the cushion to achieve the desired pillow height position.

4. Fitting the Interface

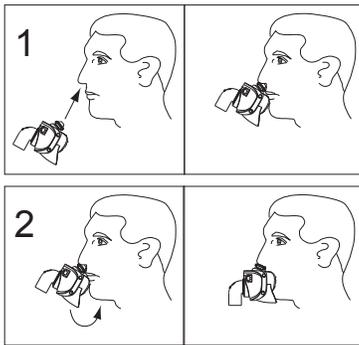

1. Insert the nasal pillows at an angle into the nose then
2. Rotate the cushion downward over the mouth.

3.




3. Confirm that the upper seal of cushion lies completely above the mouth high on the upper lip. Reposition as needed.

4.




4. Confirm the pillows are seated inside the nose and on the rim of nostril. Use fingers to guide pillows into the correct position.


5. Adjusting the Headgear

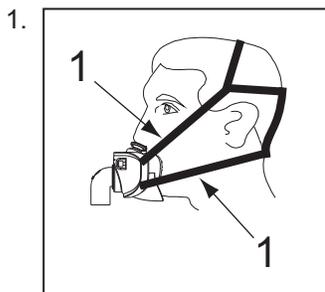

1. Gently pull each of the four side straps to achieve an overall even and balanced pressure and fit between the cushion and the face.

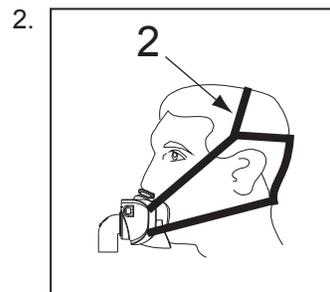

2. Adjust the two overhead straps on the headgear until the pillows fit comfortably against the nose.

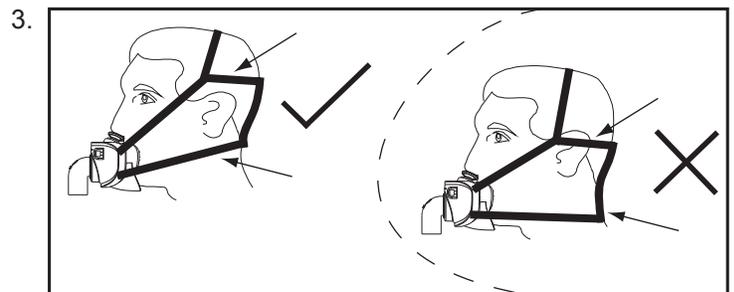

3. Confirm that the headgear is properly worn:
* The straps should not lay on the ears.
* The back portion should not lay too low on the neck.

6. Further Pillow Fitting Tips



Try the following:

1. Place pillows in tall position and repeat step 4.
2. Gently tighten the overhead strap on the headgear.
3. Try one of the other nasal pillow sizes included in the package.

Hybrid[®] AV
Hybrid[®] NE
Dual-Airway Interfaces

Sizing and Fitting Guide - 10022 Rev A

REMEMBER THE KEYS TO A GOOD FIT WITH THE HYBRID ARE:

- Proper oral cushion size
- Proper nasal pillow size
- Appropriate pillow height position
- Upper seal of cushion high on upper lip
- Headgear worn correctly
- Adjust four side straps equally
- Adjust the overhead straps



Please consult products' Instructions for Use for further instructions or contact RespCare, Inc. customer service at 800-200-9842.